

## HYDRAULIC PRESS

1. The product is for dental technician office to press an grind, which is a substitute of old-fashioned one. It is a good helper of the dentist for it's convenient, labor-saving, flexible, efficient dentists.

2. Use steps:

- 1、 Pull the pressure switch to the pressure-rising position;
- 2、 Randomly adjust the height of the platform;
- 3、 Put on the compression mold, press the pressure pole to fix the dental box;
- 4、 Use hydraulic rod to increase the pressure;
- 5、 Pull out the pressure switch to reduce the pressure, release the pressure pole to take out the box.

3. Note:

Output rate of the hydraulic system can be 15mpa to 25mpa, do not exceed the pressure range of the gauge, in order to avoid damage the pressure gauge or other system. Commonly-used pressure is 12mpa to 20mpa.

Product Name	Pressure	Use height	Characteristic
JT-LM11	15mpa	180MM	Steel plate ,tubing with brass
JT-LM11(B)	25mpa	180MM	45# steel casting.



JT-LM11



JT-LM11(B)