


JT-69&JT-69B dust cover parameters and operating instructions

This dental technician dust cover for polishing and polishing is a work of ingenuity combined with ergonomic design aesthetics according to the practical function of the technician. It is divided into two types: internal vacuum (JT-69) and external vacuum (JT-69B).

After unpacking the box, install the elbow brackets on the left and right sides with random screws according to the diagram, take out the power cord and plug it in, turn on the main switch, and the lighting and vacuum cleaner can be controlled. The nearly silent powerful vacuum cleaner and the three-way shadowless lighting design in the box allow the technician to concentrate on the work. The bottom waste box can be removed for powder treatment and then reset.

If the 69B(Dental technician dust cover (need external vacuum cleaner)) plug is powered on, just connect the Φ 50PVC pipe connected to the central vacuum cleaner to the rear pipe port of the box to start working.

	○JT-69 parameters	○ JT-69B parameters
1.Package size:55x53x35cm	<u>√</u>	<u>√</u>
weight: 16kg (18kg)		weight: 10kg
2.Product body size:50x47.5x30cm	<u>√</u>	<u>√</u>
3.Input voltage:110~240V,50~60Hz	<u>√</u>	<u>√</u>
4.Built-in vacuum cleaner		
a,Power supply voltage :(a)220V,50Hz (b)110V,60Hz(optional)		
b, maximum atmospheric pressure :15Kpa		
c, maximum vacuum :-8.8Kpa		
d, maximum noise :-55db Motor power :180W		
5.Built-in lighting		
a, Input voltage :100~240V, 50~60Hz		
b, power :14W		
c, lumen :1400lm		
d, color temperature :6500K		
e: indicates P280		

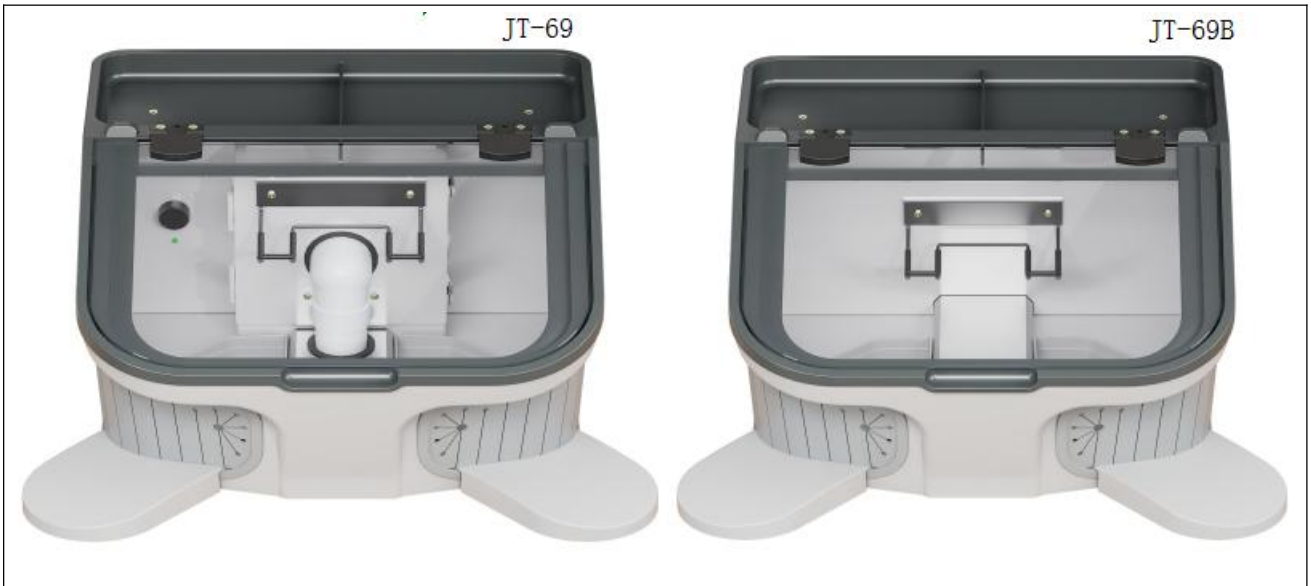

Note: Dental technician dust cover every two shifts, clean the dust collection bag, two weeks to replace the new bag. Please replace the carbon brush or replace the motor directly.